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Dane kontaktowe

XIAMEN SENYANG CO.,LTD
Address: No.189-1,Tongji South Road, Jimei District
Xiamen, 361000 Fujian P.R. China

SUNGO Europe B. V.
Address: Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

Importer

mdh Sp. z o.o.
Adres: ul. Maratońska 104, 94-007 Łódź, Polska
tel. +48 42 674 83 84, fax. +48 42 636 52 21
www.mdh.pl
www.viteacare.com

English version on next page

Table of contents

Introduction from mdh Sp. z o.o.....	21
Lift construction and technical parameters	22
Model BMW01	22
Model BMW02	23
Technical specifications.....	24
Indications and contraindications	24
Indications.....	24
Contraindications	24
Assembly	25
List of parts.....	25
Scheme of detachable electric connection	25
Lift frame assembly.....	26
Usage	28
Patient transport by lift - simulation.....	29
Patient transport to lift - different surfaces	29
Short distance lift driving	30
Seat height adjustment.....	30
User and patient safety	31
Maintenance operations and repair.....	32
Disinfection.....	32
Repair	33
Disposal of electric wheelchair.....	33
Symbols	33
Contact information	35
Importer.....	35

Introduction from mdh Sp. z o.o.

Thank you for choosing our product. We firmly believe that it will meet your expectations. Please read this document before using it for the first time. These instructions are meant to provide you with the necessary information on safe use of the device. Correct compliance with the recommendations will significantly increase the durability and aesthetics of the product. We would like to point out that we are constantly trying to improve the quality of our products, which may result in minor changes that are not included in this manual. Electric lifts are mainly assistive devices facilitating barrier-free movement of disabled people and are used for short-distance transports and rehabilitation care for people with disabilities or patients. Using a lever that controls the elevation of the seat, nursing staff can easily implement patient transports. It significantly reduces the work intensity of nursing staff and family members, improves care efficiency, reduces nursing risks and prevents patients from suffering secondary injuries during transports, while improving the quality of life and dignity of people with disabilities and promoting rehabilitation.

ATTENTION

Always observe the basic safety precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Keep this manual for future reference.

WARNING!

Improper use can lead to death, serious injury, unsafe conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the wheelchair.


SUGGESTION!


Follow the instructions below to keep your wheelchair in good working condition.

Lift construction and technical parameters

Model BMW01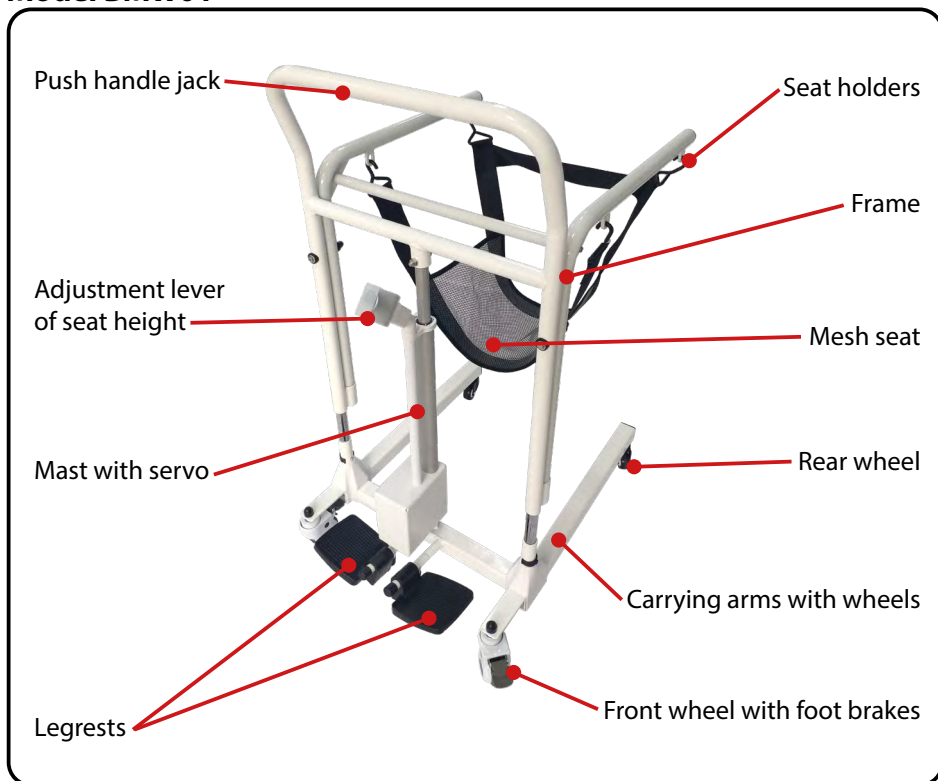

Model BMW02

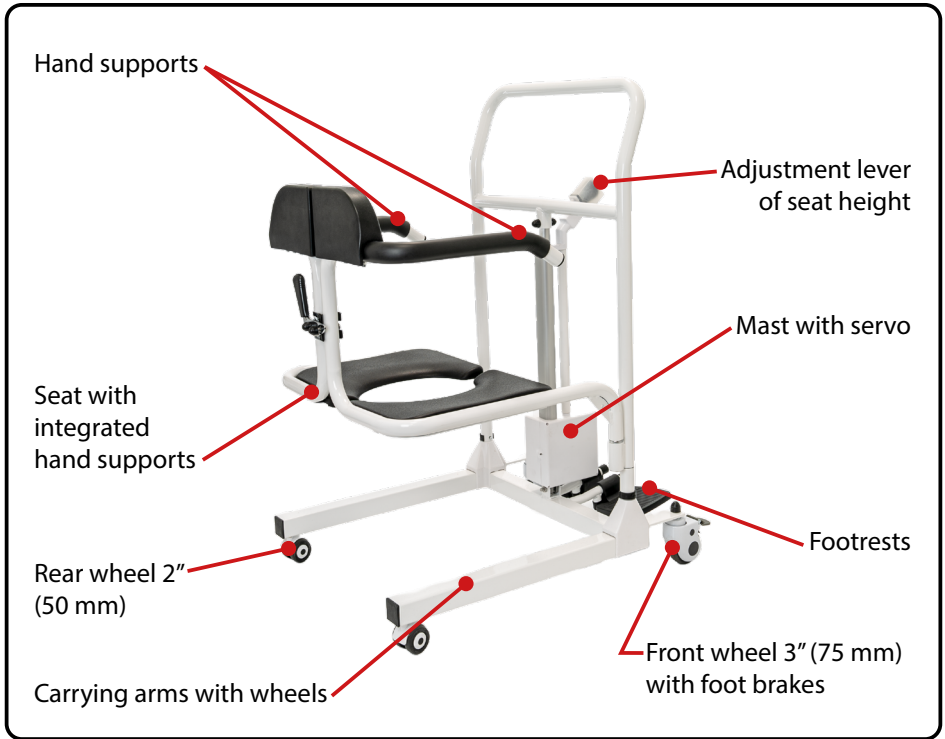Technical specifications

Parameter	BMW01	BMW02
Overall external length	750 mm	
Overall outer width	545 mm	
Overall external height	990-1240 mm	
Inner seat width	-	445 mm
Inner seat depth	-	350 mm
Range of seat height adjustment	-	250 mm
Overrun height of the seat (measured from the ground)	minimum: ~290 mm maximum: ~540 mm	minimum: 390 mm maximum: 645 mm
Height of carrying arms with wheels	110 mm	
Total height of hand supports	-	300 mm
Diameter of the front wheels	75 mm (3")	
Diameter of the rear wheels	50 mm (2")	
Weight	20,2 kg	25,6 kg
Maximum patient weight	135 kg	

Indications and contraindications

Indications

Assistive equipment is designed for people with mobility disabilities to facilitate (overrun over the toilet) or to a limited extent for indoor transportation of people with mobility disabilities.

Contraindications

Lack of independent stabilisation of the head and trunk. Contraindications for sitting, stroke, total paralysis.

WARNING!
Maximum patient weight is 135 kg.

Intended user

A medical device intended for use by a professional user.

Target patient groups

The medical device described is designed for people with mobility impairments. It is intended for people with mobility impairments to facilitate (overrunning the toilet seat) or, to a limited extent, to transport the disabled person indoors.


Assembly

After unpacking the package, check that it contains all the parts as listed below. Make sure that all parts are free from visible mechanical damage. If in doubt, contact the distributor. Do not use the unit until all parts have been properly assembled.

List of parts

Part/ element	BMW01	BMW02
Frame and servo	1 set	1 set
Seat	1 pcs	1 set
Charger	1 pcs	1 pcs
Screws and accessories	1 set	1 set
Instructions for use	1 pcs	1 pcs

Scheme of detachable electric connection

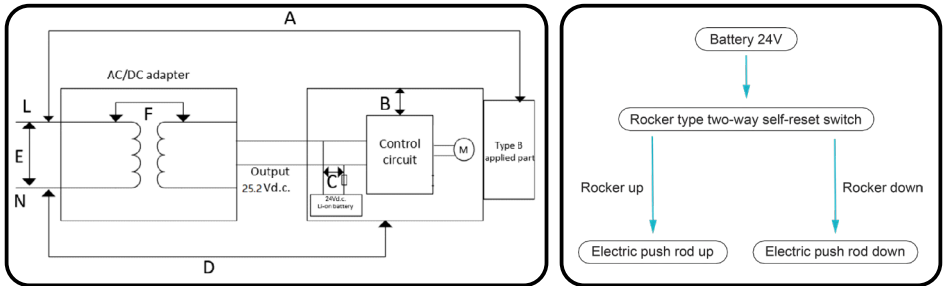
WARNING!
No modification of this equipment is permitted.


WARNING:
- Keep away from water jets
- Do not immerse in water
- Keep away from dust

Lift frame assembly


Assemble the guides.
Mount the tube guides and secure them with provided pins.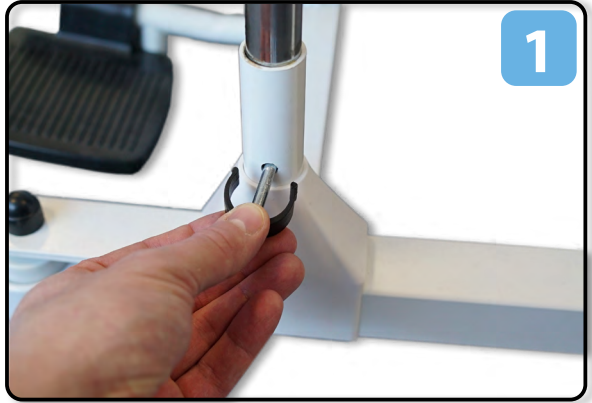

Mount the motor to the lower part of the frame and lock with the bolt with the sleeve.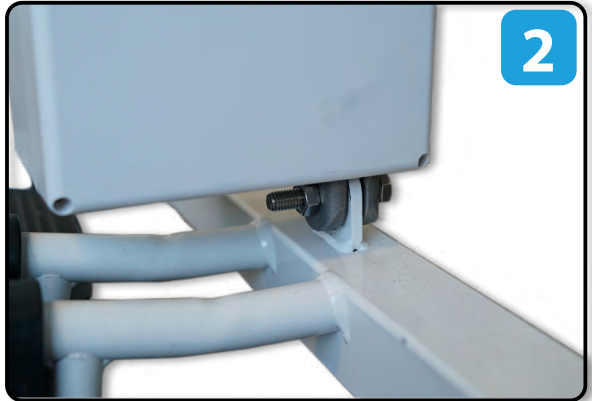

Mount the frame on the tubular slides.

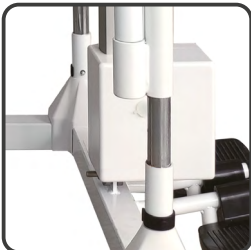
Install and tighten the tube rail entry screw at the top of the servo.

For model BMW02 - install the spring on the railing closing hook.

For model BMW02 - attach the seat to the tubular handles.

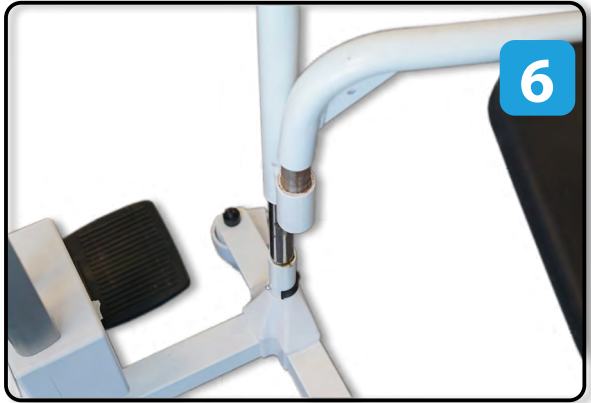ATTENTION!

If problems appear during installation, contact the distributor's service department.

Usage

The lift is a non-sterile, reusable medical device.

Before using for the first time, make sure that all components are in working order and without signs of damage. As shown in the images below, the hand supports integrated into the seat of the lift shall be provided with safeguards for the user. When transporting the user to the lift, the safety device must be released and the hand supports integrated into the lift seat should be movable so that they can be freely manipulated.


WARNING!
The seat overrun height is measured from the ground to the bottom edge of the seat - as shown in the photo opposite.

Overrun height:
minimum: 39 cm
maximum: 64,5 cm



Patient transport by lift - simulation

The following figure shows a simulation of a patient transport on a hoist. The lift with the restraints released has been brought to the patient, who is symbolized by the figure with the dotted line. Movable hand supports, integrated into the seat, make it possible to adjust the position of the lift to the patient's position without exposing the patient to undue difficulty and effort. During transport, it is possible that the patient's pelvis will need to be raised slightly so that the seat can be placed under the patient. Once the user is in a safe position on the lift, close the locking mechanism (seat lock). The lift must have the foot brakes applied each time the patient is transported.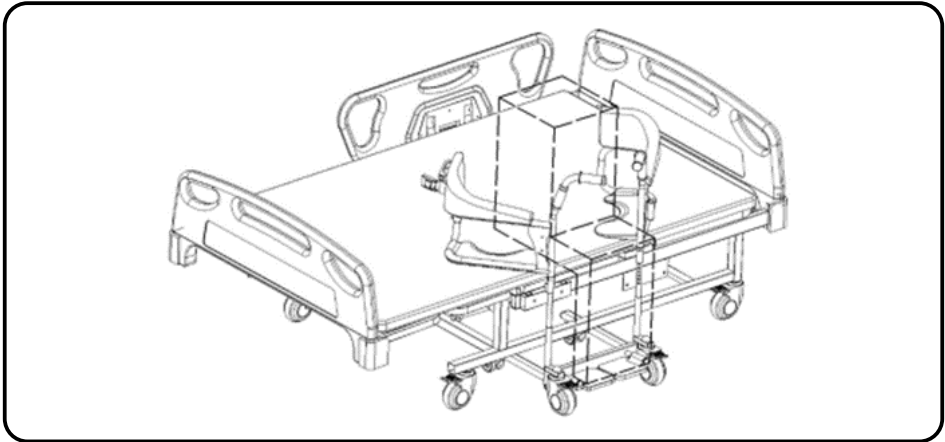

Patient transport to lift - different surfaces

The figures below show the correct positioning of the lift when transporting a patient to a given usable surface and when preparing the patient for physiological activities.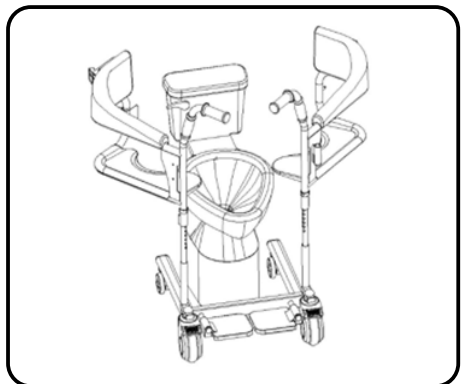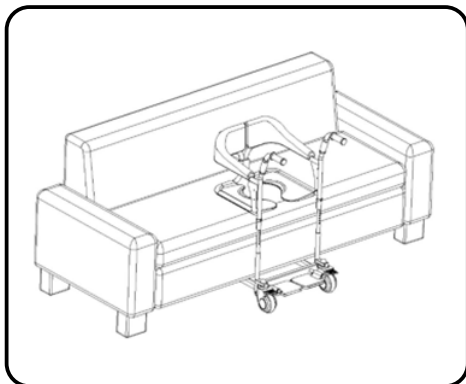

Short distance lift driving

The lift is designed for short distance patient transport. The person operating the lift should be trained in the use of the lift. For increased patient comfort, the lift is equipped with a seat cushion.


The lift must have the foot brakes applied for each transport of the patient. The brake must be released before moving the lift.

Seat height adjustment

Changing the position of the lever allows you to raise or lower the patient. Pay special attention to a correctly mounted pin (cylinder retaining screw).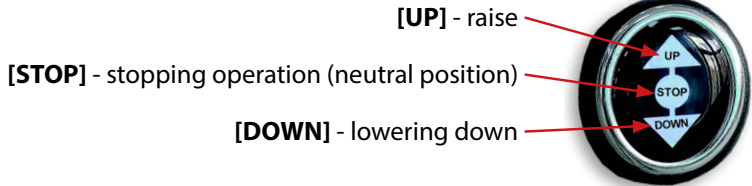

The servo will automatically stop when it is at its highest or lowest point. The electric motor operates on a low voltage supply, which has a great influence on safety.

User and patient safety

1. Please read these instructions for use before using the lift and keep them for future reference.
2. Do not step on the footrests before getting on or off the patient lift.
3. Lock the brakes when getting on or off the patient lift!
4. Do not step on the footrests before getting on or off the patient lift.
5. Do not move the lift with the user.
6. The Lift is not intended to be used to transport a patient up stairs, as a stretcher or by hoisting the patient on it. The structure of the lift and the connecting parts are not designed for such high loads if used in this way. This may weaken or damage the entire load-bearing structure and cause an accident!
7. The lift is designed for indoor use on paved and level surfaces only.
8. Only use the lift on surfaces where all four wheels of the lift touch the ground.
9. Do not leave the jack in the immediate vicinity of an open flame source.
10. Please note that the surface temperature of the seat and frame parts can increase when exposed to the sun or other heat sources, and there is a risk of the metal parts of the lift cooling very rapidly in cold weather. This can cause burns or frostbite to parts of the body.
11. The lift should be carried by a minimum of two people grasping the frame structure.
12. Use extreme caution when navigating on inclined surfaces.
13. The brakes are not intended to slow down the lift during movement and should only be used to prevent unwanted movement of the lift.
14. Do not use the lift if there is any visible or perceptible damage (e.g. breakage of the frame) as this may result in an accident.
15. Before each use, perform a visual inspection of all structural components of the product, including:
 - check the operation of the braking system. Brake by pressing the brake with your foot.
 - carefully check the stability and condition of the support frame.
16. All activities such as getting on, getting off, changing the patient's position, toileting activities ABSOLUTELY require:
 - constant assistance from the caregiver, the design of the lift does not give the patient the ability to steer and drive the lift, and to lock or unlock the brakes.
 - locking the running wheel brakes.
17. Do not leave the patient in the wheelchair (even with the brakes applied) on exits, ramps, mid-floors, etc.
18. In order to maintain the performance of the lift, it is extremely important to keep all its elements clean. Storage of the lift in very humid rooms (baths, laundries, etc.) or in the open air, combined with poor hygiene, will result in a rapid loss of the lift's functional and aesthetic parameters.
19. Do not operate the lift if there is any visible or perceptible damage, or wear on any part of the lift which indicates a potential risk of accident (e.g., wear of running wheels, cracks, etc.). Refer all repairs to qualified personnel and, during

the warranty period, only to the manufacturer's service department.

20. When transporting (e.g. in a trunk), do not put any weight on the lift, it may affect its structure.
21. Keep out of reach of children under five years of age.
22. Refer all repairs to qualified personnel of the distributor or manufacturer.
23. Use extreme caution when folding the lift, lack of attention may result in body parts getting crushed by the folding parts.

WARNING!

Any "serious incident" involving a device needs to be reported to the manufacturer and to the competent authority of the EU Member State in which the user or patient resides.

WARNING!

To avoid risk of electrical shock, charger must be connected to a power supply with protective grounding only.

Maintenance operations and repair

- Charge the battery before use.
- The battery should be recharged at least once a week or when discharged if this occurs first.
- Disinfect seat components after each use.
- Use toilet cleaners as recommended by their manufacturer's recommendations.
- Clean the lift frame with a damp cloth and the seat with mild detergents.
- On an ongoing basis, remove dirt from the lift's moving parts (wheels, bolts, clamp locks).
- In order to maintain the functional parameters of the product it is extremely important to keep all its elements clean and to adjust to the way of its storage.
- Storage in rooms with very high humidity (baths, laundries, etc.) or in the open air, combined with poor hygiene, will result in a rapid loss of performance and aesthetics.

Disinfection

If the product is used by several people (e.g. in a nursing home), commercially available disinfectants should be used. Before disinfecting, clean the upholstery and handles. Only tested and approved disinfectants may be used for disinfection.

Information on recognized and approved disinfectants and disinfection methods is available from your national health authority.

**WARNING!**

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.

Repair

If repairs are necessary, contact your distributor.

Disposal of electric wheelchair

The electric wheelchair must be disposed of in accordance with applicable local and national regulations. For information on the proper disposal of packaging, metal frame components, plastic components, electronics, batteries, neoprene, silicone and polyurethane materials, please contact your local waste disposal company or authorized dealer.

Symbols

	Medical device
	Production date
	Manufacturer
	Authorized representative in the European Community / European Union
	Importer
	Conformité Européene (compliance with the European Community)
	Catalog number (to re-order)
	Serial number
	LOT NUMBER

	<p>Unique Device Identifier</p>
	<p>Follow the instructions for use</p>
	<p>Refer to the instructions for use</p>
	<p>Warning</p>
	<p>Type B application part</p>
<p>IP44</p>	<p>Protection against objects larger than 1 mm Protection against splashing water from all directions</p>
	<p>Permitted temperature</p>
	<p>For indoor use</p>
	<p>Maximum patient weight</p>
	<p>Maximum safe inclination</p>
	<p>The product is designed to be transported by car</p>
	<p>The product is not intended to be used as a seat for transport in a motor vehicle</p>
	<p>We recommend short and thorough hand washing of the seat cushion at 30°C, do not use bleach, do not iron, do not dry clean, do not use stain removers containing solvents, do not use a tumble dryer</p>

DISPOSAL OF WASTE ELECTRICAL AND ELECTRONIC EQUIPMENT

This symbol on the products or accompanying documents means that used electrical or electronic products should not be disposed of with the normal household waste. For proper disposal, refurbishment or recycling, please take these products to the collection points for this type of waste.

Alternatively, in some EU countries or other European countries, you may return your product to your local retailer when purchasing a similar new product. By disposing of this product correctly, you will help to conserve valuable natural resources and support the prevention of potential negative effects on the environment and human health, which could result from incorrect disposal. For further information, please contact your local municipality or collection point waste collection points. Penalties may be imposed in accordance with local regulations if this product is disposed of in an inappropriate manner. For parties within the European Union, if you wish to discard electrical or electronic equipment, please obtain the necessary information from your retailer or supplier.

Elimination in countries outside the European Union. This symbol is valid in the European Union. If you wish to decommission this appliance, please obtain the correct decommissioning information from your local authorities or from your retailer.

Contact information

XIAMEN SENYANG CO.,LTD

Address: No.189-1,Tongji South Road, Jimei District
Xiamen, 361000 Fujian P.R. China

EC

REP

SUNGO Europe B. V.

Address: Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

Importer

mdh Sp. z o.o.

Address: Maratońska street 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl

www.viteacare.com

We reserve the right to make technical changes due to technical progress.

viteacare.com

mdh Sp. z o.o.
ul. Maratońska 104, 94-007 Łódź, Polska
tel. +48 42 674 83 84, fax. +48 42 636 52 21
www.mdh.pl
www.viteacare.com