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EN

Indications:

Wrist, hand and fingers disease which require support and stabilization in specific corrective and/or pain reducing positions. Tendon inflammation, carpal tunnel syndrome, rheumatic joint inflammation, contractures preventive treatment, circumferential nerves injuries, surgery.

CAUTION!

The brace should be used according to the instructions given by the doctor or physiotherapist. When used for the first time, the support should be put on by the experienced health-care personnel. Before the first use, please, read the user's manual carefully. If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

CAUTION!

Not observing the rules of personal hygiene and cleaning the orthosis with inappropriate detergents may result in skin abrasion or other effects related to sweating. Please, do not use the warming ointment.

Putting on:

Please remember to choose the proper size of orthosis to have good treatment. Put hand and forearm on the splint and fasten the Velcro belts.

1. Set the straps to most loosening setting for easier application.
2. Slide affected wrist and hand into the brace.
3. Adjust the brace to a snug fit.

Maintenance:

Orthosis can be washed by hand in a very mild detergents at temperatures up to 35°C, after removing the aluminum stay. Do not chlorinate, do not dry clean, do not wring, do not spin, squeeze gently before drying. Dry away from artificial sources of heat. Dry in exploded form. Do not dry.

Size chart:

Size	Wrist circumference (cm)	Width of the hand (cm)
S	12,5 - 15,5 cm	8 cm
M	15,5 - 17,5 cm	9 cm
L	17,5 - 20,5 cm	10 cm